

INSTRUCTIONS FOR RETURNING ORAGENE AND CONSENT FORMS



1. Make sure the cap of the oragene container is screwed on tightly.



2. Place the saliva-filled and sealed Oragene container back into the plastic case and close the lid. It may not close completely; this is okay.



3. Put bubble wrap around outside of the plastic case.



4. Place the case with the bubble wrap and the absorbent cloth inside the biohazard bag.



5. Assemble the priority mail box.



6. Fold all consent and reply forms in half. Place biohazard bag on top of folded forms.



7. Place folded forms and biohazard bag inside the priority mail box. Close and seal the box.

8. Drop the priority mail box at any accepting post office or in a priority mail drop box. Postage is already paid.

THANK YOU FOR YOUR PARTICIPATION!