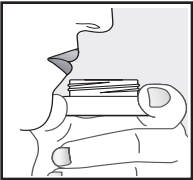
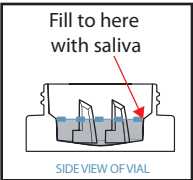
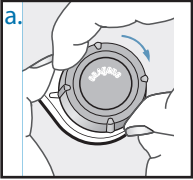
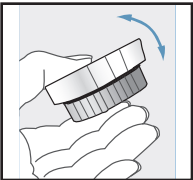




## ORAgene®•DNA Self-Collection Kit User Instructions

**Do not eat, drink, smoke or chew gum for 30 minutes before giving your saliva sample.**

### User Tips:

- Do NOT remove plastic film from the cap.
- To make more saliva, close your mouth and wiggle your tongue or rub your cheeks. Some people find it easier to spit more if they place ¼ tsp of plain white sugar on their tongue.
- Finish spitting within 30 minutes.

<p>1</p> 	<p>Start giving your saliva sample.</p>
<p>2</p> 	<p>Spit until the amount of liquid saliva (not bubbles) reaches at least the level shown in picture #2.</p>
<p>3</p> <p>a.</p> 	<p>Put the container on a flat surface. Screw the cap onto the container. Make sure that the cap is closed tightly, as shown in picture #3c. If you have trouble closing the cap all the way, turn the cap slightly counter-clockwise then try to close it again.</p>
<p>4</p> 	<p>Mix gently for at least 10 seconds.</p> <p>Recycle outer packaging.</p>

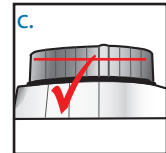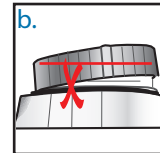

**Intended Use:** This product is designed for the safe collection of saliva samples.

**Contents:** Contains 1 collection kit. Before use, the solution in the cap should be clear and colorless.

**Warnings:** Wash with water if the Oragene•DNA liquid comes in contact with eyes or skin. Do not ingest.

**Storage:** Store at room temperature 15-30°C.